

MEMCO™



Service & Calibrations

Call Today! **570-372-0905**

112 18th Street, Selinsgrove PA 17870
(844) 636-2672

Cushion Removal / Replacement Instructions

Use your phone and take lots of pictures for reassembly!

Tools needed: Sharp utility knife, scissors, 3/8 7/16 & 1/2 sockets and Phillips bit for drill, pliers to pull stray staples out, adjustable wrench, small hammer to pound staples flat, and a good staple gun with short staples.

We recommend cutting off the old material first while the boards are bolted to the table. Next, remove one board at a time, recover it and replace it...then move onto the next one.

1. Stand table up. Remove arm rests. Remove paper cutter and holder.
2. Run front section long as possible. Remove long spring.
3. Remove pelvic notch bolt. Remove pelvic return spring. Remove 3 screws and remove pelvic cushion. Remove pelvic plate from wood side of cushion.
4. Pull ankle out and unscrew support rods.
5. Unscrew flat square hand board at front of table – 9x9.
6. Tilt head section up as far as it will go and remove all screws from the bottom of the cushion.
7. Remove the 4 screws from the thoracic piece.
8. Remove the nut from the lumbar drop-a-way latch assembly, remove the 4 screws from the slide tube, remove all the screws attaching the lumbar cushion.
9. Cut the material off the cushion with a sharp razor and remove any staples still hanging.
10. Remove 1 cushion from the table, pull the new cover over it, pull it taught and staple it, install the cushion back on the table and move to the next cushion.

Call us if you have any questions!